



24350 State Road 23 South, South Bend IN 46614

Tel: (574)233-4500

Fax: (574)288-4550

Mattresses: www.derbyindustries.com

Cases: www.derbycase.com

Email: sales @derbyindustries.com

Testing History on Cleaning Procedures For the Derby Sealed Seam Mattress

Since May 1999 Derby Industries has completed the following procedure five times a day with the same fabric to accelerate the Cleaning/Cracking Test:

Saturated fabric with Pure Bleach folded and placed weight on fabric.

The Durability of Material has sustained all testing with no signs of cracking.

We at Derby Industries strongly recommend the following cleaning procedures to ensure the durability of your quality Derby mattresses:

- *Use a hospital-grade cleaner or bleach-water solution on the entire mattress.*
- *Follow with a final rinse of water only.*
- ***After cleaning, DRY THE MATTRESS COVER THOROUGHLY . Do not place a wet mattress into bed pans, as moisture may cause fabric damage over time.***

NOTE: This damage IS NOT COVERED BY OUR WARRANTY.

Clean ability Standards DS#7566 6-20-07